

DIKANG Medical Infrared Thermometer

To change degrees from C to F:

Insert batteries.

Squeeze the trigger to turn on.

Hold down SET button, screen will display F-1, release and repeat until you screen shows F-3.

While showing F-3, use the up or down arrow to switch from C to F.

Hold down SET button to get to F-4, and then one last time to get to blank screen.

When you squeeze the trigger again the temperature will be in F.

Pressing the SET button once will change the reading from **BODY** to **SURFACE**.

You need to keep the thermometer in **BODY**.

Anytime you replace the batteries you will need to repeat the above steps.